


Sani-Cloth®
CLEANING AND DISINFECTANT

Chlor
WIPE

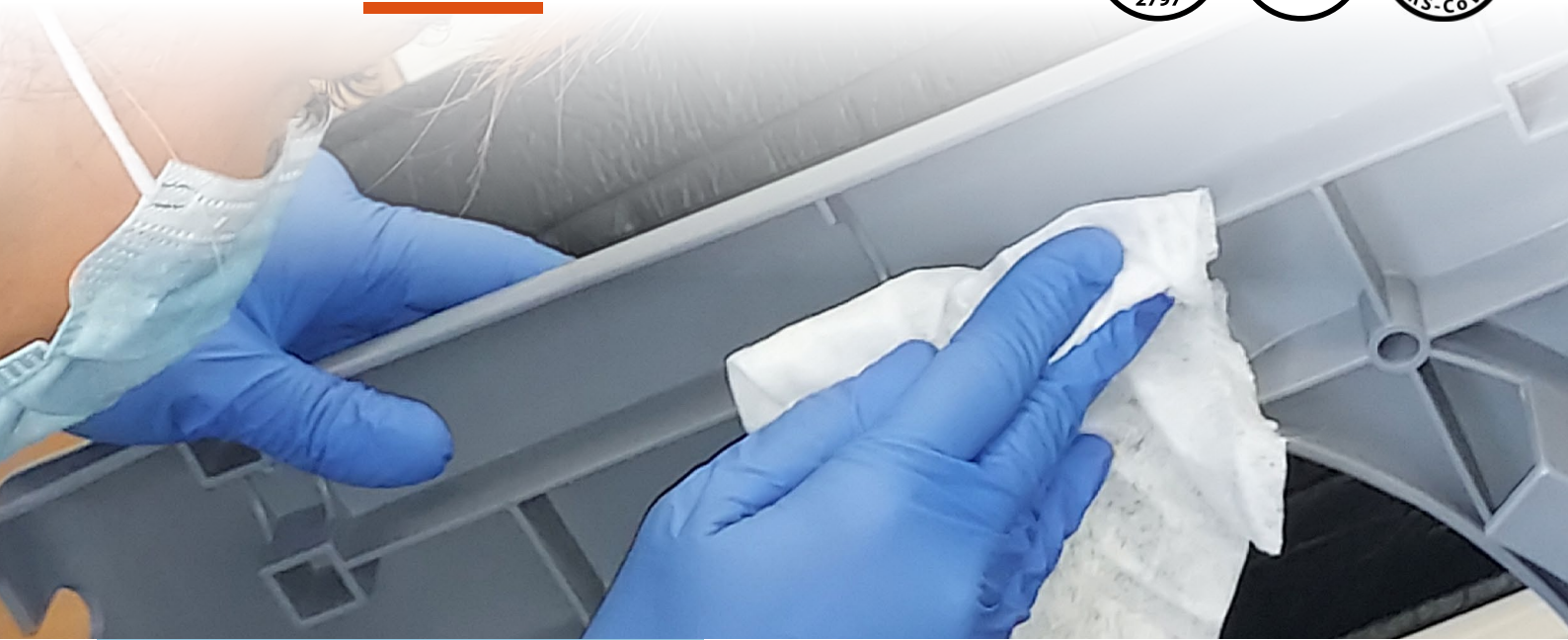

Sani-Cloth® Chlor is a sporicidal cleaning and disinfectant solution ideal for use in high risk, contaminated environments.

- + **Ready to use:** pre-dosed, low linting disposable cleaning and disinfection wipe, requiring no mixing or measuring.
- + **Proven efficacy:** effective against all spores, including C.difficile and fully virucidal in 60 seconds.
- + **Improved efficiencies:** the product solution is impregnated with sodium hypochlorite and surfactant to clean and disinfect providing you with consistently reliable results and significant time savings delivering approximately 6000 ppm available chlorine.
- + **Optimal protection:** can be used on non-invasive medical devices; non-porous hard surfaces with supporting compatibility data.
- + **Cost effective:** using the same solution for cleaning and disinfecting saves you time and money.



+ PDI
BE THE DIFFERENCE™

Environment of Care

Interventional Care

Patient Care

Product Efficacy

SANI-CLOTH® CHLOR	TEST METHOD	ORGANISM	CONTACT TIME*	
Bacteria	EN 16615	Pseudomonas aeruginosa	60 sec (c)	2 min (d)
	EN 16615	Staphylococcus aureus	60 sec (c)	2 min (d)
	EN 16615	Enterococcus hirae	60 sec (c)	2 min (d)
	EN 13727	Pseudomonas aeruginosa	30 sec (c)	
	EN 13727	Staphylococcus aureus	30 sec (c)	
	EN 13727	Enterococcus hirae	30 sec (c)	
Yeasts	EN 16615	Candida albicans	60 sec (c)	2 min (d)
	EN 13624	Candida albicans	30 sec (c)	
	EN 13624	Candida auris	30 sec (c)	
Enveloped Virus	EN 14476	Vaccinia Virus	30 sec (c)	60 sec (d)
	EN 14476	Corona,HIV, Hep B & C, SARS, Flu	30 sec (c)	60 sec (d)
Non-enveloped Virus	EN 14476	Norovirus	60 sec (c)	60 sec (d)
	EN 14476	Adenovirus	60 sec (c)	
	EN 14476	Poliovirus	60 sec (c)	
Spores	EN 16615	C.difficile	60 sec (c)	
	EN 13704	Bacillus subtilis	5 min (c)	
	EN 17126	C.difficile	15 min (c)	
	EN 17126	Bacillus subtilis	15 min (c)	
	EN 17126	Bacillus cereus	15 min (c)	

*Clean Conditions (c) Dirty Conditions (d)

Point of Use Accessories



Bracket for canisters

Bracket (colour coded): **A001ORG**

Canister brackets affix securely to the wall, holding the canister in place at its location.



Pole Adapter

Product Code: **A002POL**

Pole brackets make it possible to affix the canister brackets on to portable equipment such as BP stands.



I Am Clean Sticky Notes

Product Code: **A005SNT**

Recyclable sticky notes which can be used to easily and effectively identify equipment or surfaces that have been cleaned.



Educational wall charts and signage

Product evaluations, audits, educational wall charts & signage, that can be personalised for your specific needs.

Range Details

	REORDER NO	WIPE COUNT	WIPE SIZE	QTY PER CASE	CLASSIFICATION
50ct Large Canister	ECH1400	50	200mm x 195mm	6	Medical Device Class IIa

Other formulations available:



Find out more about how our Layered Approach to Infection Prevention can help you.

PDI International, Pywell Road, Willowbrook Est. Industrial Estate, Corby - Northants, NN17 5XJ, UK

+44 (0) 8081 697 945 contact@pdi-emea.com wearepdi-intl.com PDI International PDI_Intl

+PDI
BE THE DIFFERENCE™